Cat. no. 78/022 ms001

GB - Canpol babies Hot & Cold Therapy Gel Postpartum Pads 2 pcs. Reusable gel pads in the form of compresses support regeneration in the postpartum period, e.g. they relieve perineal pain after childbirth, hemorrhoid problems and provide relief from discomfort after a cesarean section. They can be used with or cold depending on your needs. They are hygienic and safe. Without BPA. The package contains 2 reusable gel pads and 12 pieces of replaceable disposable covers for comfort and hygiene after childbirth.

General precautions
1. The product is intended for external use only.
2. Do not pierce the package with sharp objects or apply excessive pressure to it. Do not hit or step on the package.
3. In the event of damage, tearing or cracking of the gel compress, stop using it and throw it in the trash.

4. In the event of cracking of the product, avoid contact with eyes and skin. In the event of contact of the liquid with eyes or skin, rinse the area thoroughly with clean water.
5. If the contents of the gel compress are accidentally swallowed, drink plenty of water, try to vomit the stomach contents and contact a doctor if necessary.
6. Do not apply directly to bleeding skin or in the case of injuries with inflammation.
7. Do not apply hot/cold gel compresses directly to the skin, especially straight from the microwave or refrigeration.
8. If you have circulation problems, consult a doctor before use.
9. Regularly clean the gel compresses in lukewarm water with a mild detergent. Leave to dry.
10. Store in a dry and cool place.
11. The product is not a toy, keep out of the reach of children.
12. If you have any health concerns, consult a doctor.

How to use?

HOT COMPRESS

Warm compresses can help relieve discomfort associated with uterine contractions, providing soothing and pain relief. They can also be used to relieve chronic back pain, headaches, joint pain, and tension in the neck and shoulders. The heat helps relax muscles and improves blood circulation, which can help reduce discomfort.

Heating in a microwave

Place one gel pad in the microwave, laying it flat so that the gel content inside is evenly distributed. NOTE Never heat a folded/ crumpled gel pad. Each compress should be heated separately. Place the gel pad in the microwave (40 seconds at 385W, 25 seconds at 600W). Always check the temperature of the compress before use. Heat the pad to the temperature you feel is right. Once heated, the pad should feel warm to the touch, but not hot. If necessary, increase the heating time by an additional 5 seconds. Do not overheat. Before applying it to the body, dry the gel insert, e.g. with a towel, and place it in a disposable cover to protect the skin from burns. If you do not have a cover, you can wrap it in a towel or a cotton/muslin/cotton diaper. Then apply a warm compress to the painful area until you feel discomfort.

• Heating in a water bath
Fill a large container with water so that the insert is completely submerged, without touching the bottom of the container. Bring the water to a boil, then remove the container from the heating surface. Do not submerge the insert while the water is still heating. Use tongs, for example, to immerse and remove the insert from the water. Do not leave the insert unattended while heating. Immerse the insert for 2-3 minutes. If it has not reached the desired temperature, immerse it again in the same water for an additional minute. Do not allow the product to come into prolonged contact with a vessel or other direct heat source, as this may damage it. Before applying it to the body, dry the gel insert, e.g. with a towel, and place it in a disposable cover to protect the skin from burns. If you do not have a cover, you can wrap it in a towel or a cotton/muslin/cotton diaper.

WARNING: Before each use, check the product for damage or leaks. If damage is found, discontinue use.

COLD COMPRESS
Cold compresses have a soothing and analgesic effect on pain caused by a tear in the perineum or hemorrhoids and provide relief after a cesarean section. Cold therapy constricts tissue and blood vessels, which helps relieve pain and swelling.
• Cooling in the freezer
Place the gel pad in the freezer at -18°C/-0.4°F for around 2 hours. Before applying to the body, dry the gel pad, e.g. with a towel and place it in a disposable cover to protect the skin from frostbite. If you do not have a cover, you can wrap it in a towel or a cotton/muslin/cotton diaper. Then apply a cold compress to the selected area of the body until you feel discomfort. A single application of the compress should not last longer than 20 minutes. Do not use a cold product longer than recommended.

WARNING: Do not warm up a compress taken from the freezer or place a warm compress in the freezer as this may damage the product. Always check the temperature of the product before use. Store in a dry and cold place after each use.

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