Cat. no. 78/023 ms001

GB • Canpol babies Hot & Cold Therapy Gel Breast Packs 2 pcs. Soft, reusable gel compresses. Their large diameter and flexible design make them fit perfectly to the breast without irritating the nipple. They are hygienic and safe. Without BPA.

General precautions

- The product is intended for external use only.
- Do not pierce the package with sharp objects or apply excessive pressure to it. Do not hit or step on the package.
- In the event of damage, tearing or cracking of the gel compress, stop using it and throw it in the trash.
- In the event of cracking of the product, avoid contact with eyes and skin. In the event of contact of the liquid with eyes or skin, rinse the area thoroughly with clean water.
- If the contents of the gel compress are accidentally swallowed, drink plenty of water, try to vomit the stomach contents and contact a doctor if necessary.
- Do not apply directly to bleeding skin or in the case of injuries with inflammation.
- If you have circulation problems, consult a doctor before use.
- Regularly clean the gel compresses in lukewarm water with a mild detergent. Leave to dry.
- Store in a dry and cool place.
- The product is not a toy, keep out of the reach of children.
- If you have any health concerns, consult a doctor.

How to use?

HOT COMPRESS

Compresses can be used before or during breastfeeding or expressing milk with a breast pump, relieving breast pain and swelling and supporting the flow of milk, which in turn shortens the pumping time. They also help unblock clogged milk ducts, facilitating the flow of milk.

Heating in a microwave

Place one gel pad in the microwave, laying it flat so that the gel content inside is evenly distributed. NOTE: Never heat a folded/ crumpled gel pad. Each compress should be heated separately. Place the gel pad in the microwave (40 seconds at 385W, 25 seconds at 600W). Always check the temperature of the compress before use. Heat the pad to the temperature you feel is right. Once heated, the pad should feel warm to the touch, but not hot. If necessary, increase the heating time by an additional 5 seconds. Do not overheat. A single application of the compress should not last longer than 20 minutes. Do not use a warm product longer than recommended.

WARNING: Before each use, check the product for damage or leaks. If damage is found, discontinue use.

COLD COMPRESS

Cold therapy brings relief to sore and swollen breasts, but also helps relieve pain and swelling in muscles and joints and lower body temperature.

Cooling in the freezer

Place the gel pad in freezer at -18°C/-0.4°F for around 2 hours. Always check the temperature of the product before use. Then apply a cold compress to the selected area of the body until you feel discomfort. A single application of the compress should not last longer than 20 minutes. Do not use a cold product longer than recommended.

WARNING: Do not warm up a compress taken from the freezer or place a warm compress in the freezer as this may damage the product. Always check the temperature of the product before use. Store in a dry and cold place after each use.

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