GB • Soft hydrogel breast pads provide soothing relief to your sore nipples, support skin regeneration and restores the skin's natural moisture. Hydrogel pads also provide extra cushioning and protection from clothing.

Instructions for use:

- Wash hands before using breast pads.
- Open package, remove hydrogel pad from sachet and remove protective backing. Avoid touching the moisturized side with your fingers.
- Gently place moisturized side on nipple skin.
- Prior to breastfeeding, remove the pads and place on a clean surface, with the moisturized side up.
- Gently wash nipples with lukewarm water and pat dry.
- If the hydrogel pad becomes cloudy or dirty, throw it away and use a new one. Cleaning may result in loss of product functionality.
- Unopened sachet of hydrogel pads can be placed in the refrigerator shortly before use for even better cooling.

Warnings:

- Use only as indicated. Keep out of reach of children.
- Store in a cool, dry place.
- Do not use if the package is opened or damaged.
- Particular caution is recommended when applying to severely cracked nipples. The hydrogel insert is not intended for the treatment of deep wounds or bleeding wounds.
- If nipples are bleeding or have a burning sensation, contact breastfeeding specialist, midwife or medical doctor.

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